

# ClearTemp™ LC



Unlike other temporary cements, **ClearTemp LC cement** is designed specifically for temporary veneers. Its proprietary, light-cured resin formula provides the additional strength required to keep provisional veneers in place. The translucent shade won't show through temporary veneers, and unique fluorescing properties cause ClearTemp LC cement to glow under a black light for easy detection. The cement will also adhere more to the provisional than to the tooth, minimizing the time and effort required to remove it from the preparation. For luting temporary veneers, nothing will hold as strong or look as natural as ClearTemp LC cement.

**STRENGTH THAT'S TEMPORARY**  
for a look that's not

# ClearTemp™ LC

- Translucent shade is ideal for temporary anterior veneers
- Light-cured resin formula provides a quality seal and exceptional retention
- Fluoresces under a black light facilitating complete removal
- Ergonomic contra-angle syringe delivery aids in precise placement



ESTHETIC

Today's provisionals look more natural than ever. ClearTemp LC cement helps to create a short-term smile that patients will be proud to reveal.



COMPARE

A traditional temporary cement shows through the provisional crown on #8. ClearTemp LC cement does not show through the provisional veneer on #9.




FLUORESCING PROPERTIES

ClearTemp LC cement fluoresces under black light for easy detection. Use black light to ensure complete removal of ClearTemp LC cement. This is an important step that minimizes potential to damage final restoration. Use the VALO™ black light lens attachment or UltraSeal™ XT hydro black light keychain for high visibility.

Note: Always use the VALO curing light with the barrier sleeves during procedure.

## Instructions

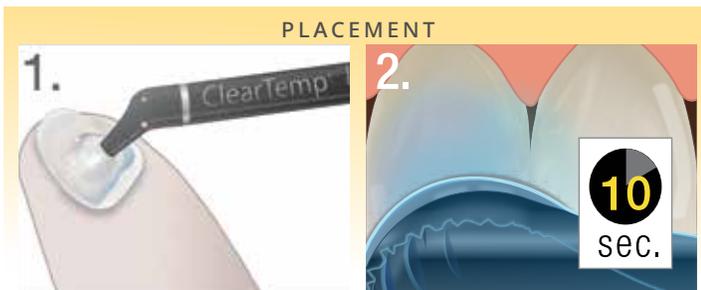

PLACEMENT



Remove product from refrigerator and bring to room temperature. Clean, rinse, and lightly dry the preparation. Express enough ClearTemp LC cement to coat the inside surface of the provisional.




Seat temporary veneer on the preparation and remove flash. Light cure with VALO™ curing light on Standard Power mode for 10 seconds.



REMOVAL



Use a hand instrument at the acrylic margin to break the seal and remove provisional. ClearTemp LC cement is very strong and has high adhesion, so temporary veneers may break upon removal. Flake off bulk residual cement with a blunt hand instrument.

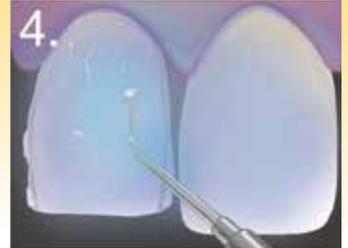

Illuminate surface with black light to reveal remaining ClearTemp LC cement. Remove any remaining cement and recheck. Scour prep with pumice-type slurry and cup or brush. Rinse thoroughly and prepare for final cementation.

**Note:** Due to its high bond strength compared to other temporary cements, ClearTemp LC cement should be used for temporary veneers ONLY.



3518 - ClearTemp LC Syringe 4pk  
0.67 g syringes

We recommend  
**PermaShade™ LC**  
veneer cement for luting  
permanent veneers

